
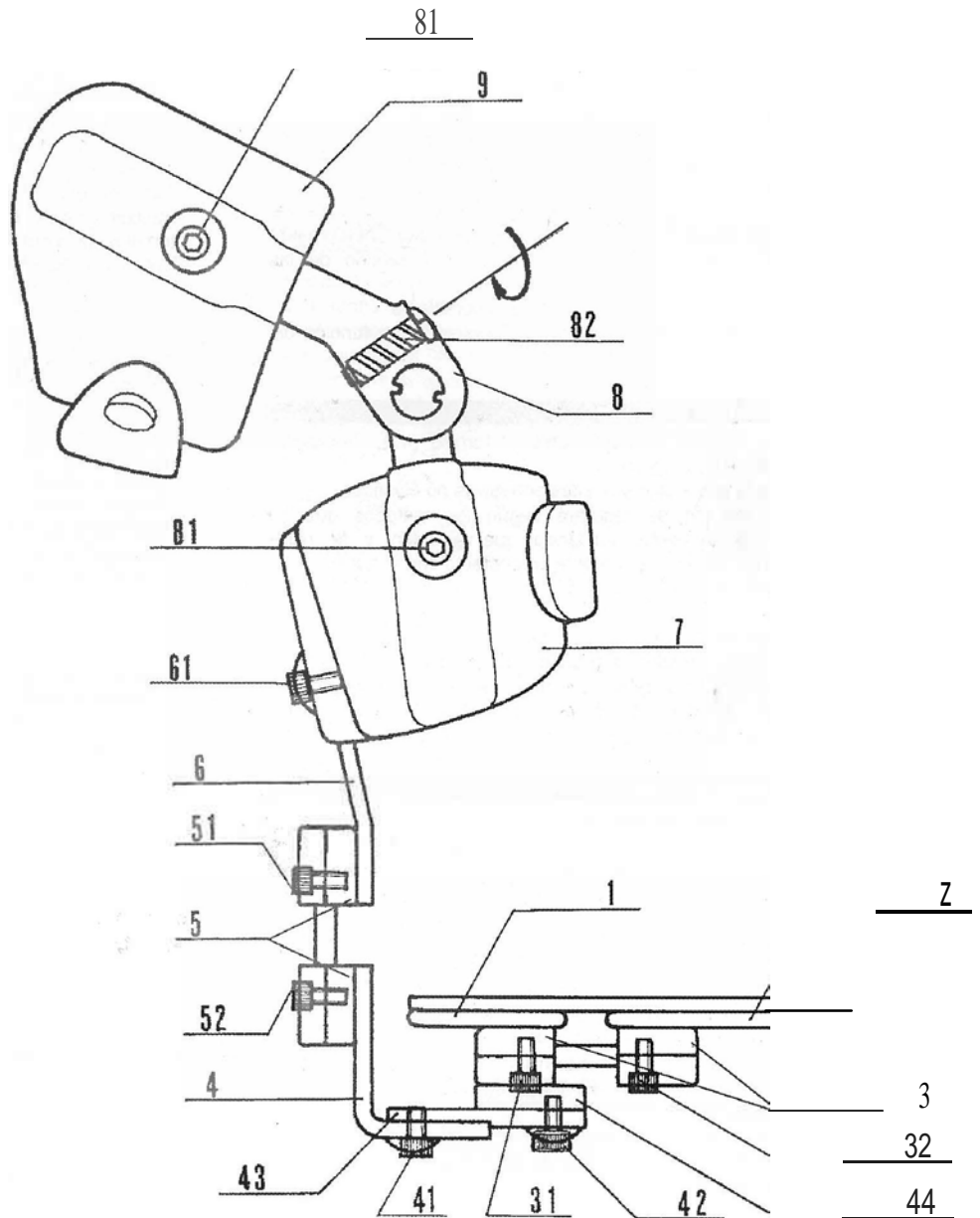
Clubax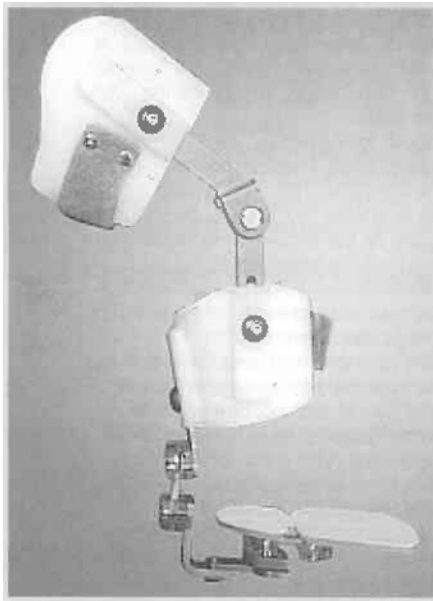

CE

ENG - Instructions of Use Orthosis Clubax

Carefully read these instructions before using the orthosis.

Indications

First intention treatment for the talipes equinovarus, pes varus, pes supinatus, pes talus valgus.

Fitting instructions

1. Positioning the cuffs

Slightly unscrew the screws (61) and (81). Put the splint on and adjust the position of the cuffs (7-9) and the heel set (4) according to the morphology of the newborn's lower limb. This adjustment must be realized knee flexed in 80°-90°. When the correct position is obtained, tighten the screws (61) and (81).

The axis of the knee flexion part (8) must be aligned to the axis of the child's knee flexion. If you use the mobile part, realize a maximum passive flexion and check that the cuffs do not pinch the soft parts on a part with the hollow popliteal. If you use an adjustable knee part, realize this passive flexion using the Allen key which you will have introduced into the screw (82), by turning clockwise. In the event of pinching, re-adjust both cuffs. During visits, do not forget to adjust the cuffs according to the growth of the child.

2. Fixing the foot on the sole

After having completely unscrewed the screw (42), remove the sole set. Fix the foot on the sole using non elastic adhesive bands. The fixing of the foot will be realized according to the methods normally practiced taking care that the heel and the forefoot are perfectly in contact with the sole. In all cases never cover the screws (31-32) of the multidirectional hinge. Replace and position the sole on the heel set (43) using the screw (42) and adjust the desired position according to the disorder to be treated and its various deformity components. When the correction position is obtained, tighten all the screws.

3. Possible corrections

The varus and the valgus of the hind foot. Unscrew the screws (51-52) of the multidirectional hinge(5) and incline the heel set (4) towards the left or the right. When the desired correction position is obtained, tighten the screws.

The abduction and total adduction of the foot

Unscrew the screw (41) and adjust the position of the foot to align the total global rotation of the foot in relation to the vertical axis of the tibial diaphysis by moving forward or backward the slider part (43) then tighten the screw (41). Unscrew, without removing it, the screw (42), slightly raise the sole set (1-2-3-44) and turn it towards the right or the left. When the desired correction position is obtained, replace the sole set in its Lodging and tighten the screw (42).

The equinus

Unscrew the screw (61) and the screws (51-52) of the multi-directional hinge (5), pull the set (6-5-4) vertically downwards in order to stretch the Achilles tendon. Simultaneously realize a dorsal flexion of the foot. When the desired correction position is obtained, tighten the screws (61) and (51-52) of the part (5).

Abduction and adduction of the forefoot

Unscrew the screws (31-32) of the part (3) and realize an abduction or an adduction of the forefoot in relation to the hind foot. When the

desired correction position is obtained, tighten the screws of the part (3).

Pronation and supination of the forefoot

Unscrew the screws (31- 32) of part (3) and realize a pronation or supination of the forefoot in relation to the hind foot. When the desired correction position is obtained, tighten the screws of the part (3).

4. Positioning a Bébox

Completely unscrew the screw (61) and withdraw the foot set from its lodging. Completely unscrew the screws (51-52) of the multi-directional hinge (5). Lift off the sole set by completely unscrewing the screw (42). Completely unscrew both screws (31) in order to dissociate the metal part (44) from the sole. Replace the short Achilles part (6) by the long Achilles part (60) delivered separately with the splint.

Replace the short multidirectional hinge (5) by the long multidirectional hinge (50) delivered separately with the splint. Fix the metal part (44) on the posterior part of the multidirectional hinge situated under the Bébox orthosis using the screws of the splint (31). Replace the foot set in its Lodging and tighten the screw (61). The splint is ready for the corrections.

Cautions

Always respect these instructions of use and the particular indications prescribed by your doctor. A qualified Medical Practitioner is the only person who can prescribe and decide the duration of the treatment. For a higher efficiency, follow the instructions specified by your doctor regarding the daily time of use. For complete therapeutic results, you must use the correct size, which first fitting should be done by a qualified person. Once fitted, do not try to modify the adjustment of the orthosis. Your doctor, during the consultations, will increase or reduce the correction of the orthosis. He is the only person who can decide the end of the treatment. For any doubt, always consult your doctor or the person who fitted the orthosis. If you notice any secondary effect, inform your doctor immediately. This orthosis should never be used for any different use other than prescribed. Do not allow children to play with this orthosis. This orthosis may not be used by another patient. This orthosis can be used with Bébox bootee (optional, not included/consult sizing). The convenience of the orthosis has to be inspected periodically by a responsible adult. Do not use creams during the treatment in order to avoid its deterioration. The child does not have to wear the orthosis when bathing or having a shower, etc.

Maintenance instruction and lifetime

Clean with a wet cloth with lukewarm water. Do not bleach. Dry with a dry cloth. Under normal conditions of use, lifetime of this orthosis is of 6 months.

Classification

Medical Device, Class I, non-sterile, with no measurement function. This product fulfils the Medical Device Directive 93/42/CEE, by which the medical products are regulated.

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