

Appendix - Anhang

ENG

Effective immediately all SPL Knee joints and SPL C.O.S.™ KAFO'S are being executed with the new modified Satellite. The Satellite has the following features:

- Automatic locking (Mode 1)
- Single unlocking (Mode 2)
- **Permanent unlocking (New)** (Mode 3)
- Permanent locking (Mode 4)

NOTE

By deactivating the permanent unlocking mode the Satellite switches into the permanent locking mode and needs to be pushed into the Automatic function.

NED

Per direct worden alle SPL scharnieren en SPL C.O.S.™ orthesen uitgevoerd met de gemodificeerde uitvoering van de satelliet. Deze heeft de volgende functies:

- automatische vergrendeling (Mode 1)
- eenmalige ontgrendeling (Mode 2)
- **permanente ontgrendeling (Nieuw)** (Mode 3)
- permanente vergrendeling (Mode 4)

LET OP

Bij het deactiveren van de permanente ontgrendeling springt de satelliet eerst in de permanente vergrendeling modus en dient daardoor omhoog in de automatische vergrendeling gedruwd te worden.

DE

ab sofort werden alle SPL Kniegelenke und SPL C.O.S.™-Orthesen mit der neuen Ausführung des Satelliten ausgeliefert. Dieser hat folgende Funktionen:

- automatische Verriegelung (Mode 1)
- einmalige Entriegelung (Mode 2)
- **permanente Entriegelung (Neu)** (Mode 3)
- permanente Verriegelung (Mode 4)

ACHTUNG


Beim Lösen der permanenten Entriegelung springt der Satellit in die permanente Verriegelung und muss wieder aus der Stellung in die Stellung automatische Verriegelung gebracht werden.




Mode 1



Mode 2

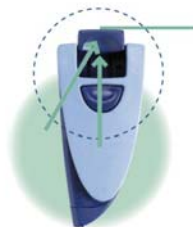

Mode 3

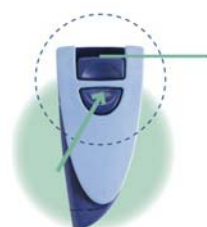

Mode 4