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● Machine wash, warm water. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on flat surface. Do not hang-dry. We recomend that you use the mesh bag, which you received with your product to protect it during washing. Close all Velcro® straps into the fabric prior to laundering to prevent exposed hooks from damaging other materials, and to prevent lint or dirt from accumulating in the hook.

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Release all Velcro® straps. Place the back support at the required height and without applying tension, wrap it to the front and close.

Release all Velcro® straps. Place the back support at the correct height. Adjust the back support at the sides so that the front panel lifts the abdomen into a comfortable position to relieve stress on your back, then close the

Grasp the lower straps and apply an even amount of tension as you pull them toward the front.

Overlap and adhere the left strap onto the top of the right strap and press the underneath Velcro® hook on the right strap into the pile fabric of the back support.

Repeat the same procedure as shown at left for the lower straps.

Pull the long straps over your shoulders, applying just enough tension so the upper back section of the back support contours to your back.

Bring the straps under your underarms and cross in the back.

Pull the straps from the rear to the front, applying enough tension to provide the required positioning and support for the upper back. Maintain even tension as you close the straps, pressing the left strap over and into the right strap. You can close them over either the upper or lower back support

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IU.12.1 © 2007 Prim, S.A.