NAVIGAIT

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ENGLISH

NAVIGAIT

Carefully read this instruction sheet before use!

Intended use and indications
Navigait is intended to support a foot when the ability to actively dorsiflex is reduced due to conditions such as Stroke, Multiple sclerosis, Post polio syndrome, Spinal Cord Injury.

Contraindications
Navigait should not be used by patients who present with spasticity or severe edema.

Limitations
Navigait will not manage misalignment of the foot or foot deformities. These should be managed with appropriate shoes and/or orthotics.

Adjustments
Adjustments to this product may only be done by professional healthcare providers. When needed, the plastic frame may be trimmed to size.

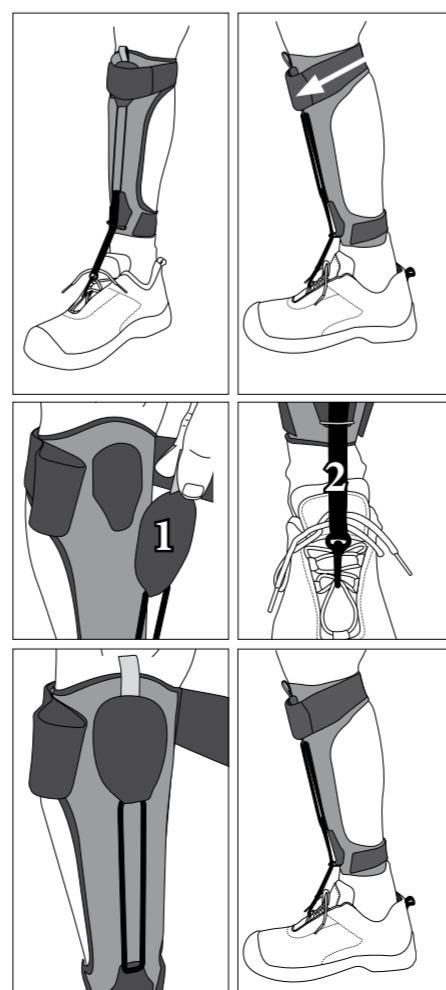
Warning
Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation. Remove the brace immediately if you notice any changes in skin color or if you notice any tingling sensations, aching or chafing feeling. Check skin on a daily basis.

Material Specifications
Frame: Polyethylene
Padding: Outside material 100% polyester. Core material polyurethane.
Elastic Coil: Rubber, contains Latex.
This product is not flame retardant.

Washing Instructions
Padding & Straps: Remove the padding and straps from the frame. Close all Velcro into the material prior to laundering. Machine washable in 40°C (104°F) gentle cycle. Use a mild detergent. Dry flat. Do not tumble dry or dry clean.

Plastic Frame:
The plastic frame can be rinsed in lukewarm water and cleaned with a mild soap. (Remove the elastic strap before cleaning). All parts must be dry before use!

Who to contact
Please contact your healthcare professional immediately in case of any discomfort, skin discoloration or irritation, or any signs of circulation problems.

Instructions for use

1. Check that the support is complete and without damage.

2. Position the brace so that the upper strap sits on the head of the calf muscle. Apply the strap at an angle from back to front as shown in the illustration. Leave open in front. The lower edge of the brace should be seated comfortably above the ankle bones. If needed the brace can be trimmed slightly in the upper and lower sections with scissors.

3. Adjust the tension (how much lifting affect you need) by either (1) moving the top velcro parts, or (2) adjusting how far down the hook on the tension strap is located on the shoe lacing.

4. Secure the strap over the pad around the leg.

Care of your NavigAIT
When the brace is not being used, roll the plastic frame longitudinally. Wrap the straps around the top and the bottom to secure. This procedure will keep the plastic frame pre-shaped.

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