Basko

Healthcare

P.E.A.care

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Instructions for Use

Your doctor will prescribe a P.E.A. **care** Splint and determines the duration of treatment. The medical instructions on daily usage must be followed exactly. External medications should be used under medical supervision only. The technician fitting the Splint is responsible for the correct fitting and functioning. In the event of skin irritation, abnormal pain or any other complication you believe to be related to the product, please contact your doctor and/or technician.

Product Scope/Intended use / Indications

The use of a P.E.A. **care** Splint is indicated for the treatment of club foot inter alia according to the Ponseti method. The P.E.A. **care** Splint is an orthotic device. The complete orthotic device consists of a P.E.A. **care** Splint with Mendivil orthotic shoes attached to its ends. When using the orthosis, the feet will be kept in the necessary external rotation position and at shoulder width. This will retain the foot correction achieved during preliminary treatment and will help prevent relapse.

Application is indicated for:

- Pes equinovarus congenitus (ICD10: Q66.0)

Permitted conditions of use / Application Points

Splints may not be used in salt water.

Risks associated with use and contraindications

The splint may not be worn over pressure points or irritated skin. Otherwise there are no known risks associated with use and no contraindications.

Maintenance Instructions

Check the Splint regularly for wear around the fittings and quick release mechanism.

Information on Materials Used

Bar and foot plate: Aluminium.

Cleaning/Disinfection Instructions

Clean the Splints with a cloth and warm water to a max. 40°C. Do not dry-clean. Do not tumble dry.

Fitting Instructions for P.E.A.care Static, Static Quick Release, Dynamic and Dynamic Quick Release

First check the contents of the package:

1. P.E.A.care Splint
2. Plastic bag with 2 Allen keys and 4 screws
3. Professional instructions
4. Information for parents/caregivers

Assembling the Splint

Step 1

Measure the child's shoulder width (**Fig.1**) and determine the correct Splint size (see size chart).

Step 2

Setting the Splint Width: Loosen the set screws with the Allen key (**Fig.2a**). Turn the Splint over and set it at shoulder width (centre to centre) (**Fig.2b**). Position the set screws as far from each other as possible and tighten the screws.

Step 3

Loosen the foot plate screws with the Allen key and remove the foot plates (**Fig.3**).

Step 4

If you want to set a dorsal flexion angle (according to the Ponseti method), order the P.E.A. **care** dorsal flexion component (separately available) **(Fig.4)**.

Step 5

Assemble the (external rotation) footplates with the supplied screws onto the sole of the (Mendivil) orthotic shoes **(Fig.5)**.

Warning: *The angle scale must always be positioned laterally to the shoe.*

Step 6

Setting the Abduction Angle: The foot plate has There is an angle scale on the foot plates. Position the foot plates at the relevant desired required angle to on the base bar using the marking in the bar to align the footplate and align this with the markings on the base **(Fig.6)**. Now screw the foot plate to the base bar.

Step 7

Repeat step 6 on the other side of the Splint.

Step 8 (only for Static Quick Release and Dynamic Quick Release)

Check the function of the quick release mechanism. Slide the shoe with the foot plate laterally onto the footplate adapter. Press and click the foot plate onto the adapter. **(Fig.7)**.

To release, squeeze the red releases latches together and push the shoe with the foot plate away from the Splint.


P.E.A.care "Static"

Spare parts

Nr.	Item description	Art. nr.
1	Base bar	2802-1 /-2
2	Footplates	2801
3	Set screws (M5 x 6)	2826
4	Footplate screws (M5 x 8)	2827
5	Shoe mounting screws (M6 x 10)	2825

Accessories

Nr.	Item description	Art. nr.
6	Quick Release-Kit	2819
7	Dorsal flexion component	2828

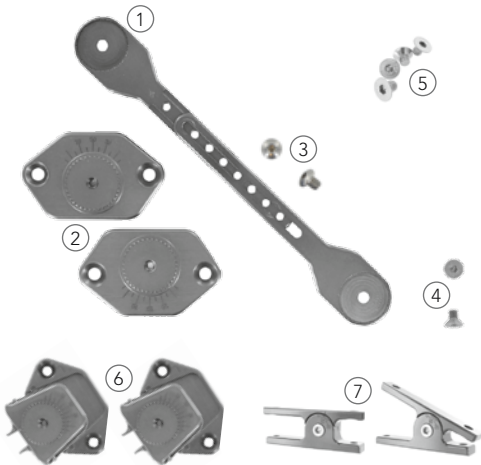P.E.A.care "Static Quick Release"

Spare parts

Nr.	Item description	Art. nr.
1	Base Bar	2802-1 /-2
2	Footplate connectors	2824
3	Footplates	2823
4	Set screws (M5 x 6)	2826
5	Footplate screws (M5 x 8)	2827
6	Shoe mounting screws (M6 x 10)	2825

Accessories

Nr.	Item description	Art. nr.
7	Dorsal flexion component	2828


P.E.A.care "Dynamic"

Spare parts

Nr.	Item description	Art. nr.
1	Base Bar	2831-1 /-2
3	Footplates	2801
4	Set screws (M5 x 6)	2826
5	Footplate screws (M5 x 8)	2827
6	Shoe mounting screws (M6 x 10)	2825

Accessories

Nr.	Item description	Art. nr.
6	Quick Release-Kit	2819
7	Dorsal flexion component	2828

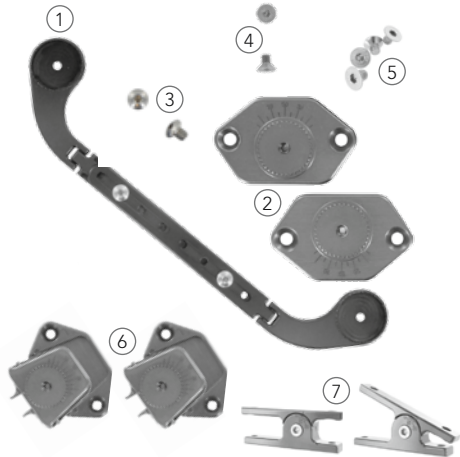P.E.A.care "Dynamic Quick Release"

Spare parts


Nr.	Item description	Art. nr.
1	Base Bar	2831-1 /-2
2	Footplate connectors	2824
3	Footplates	2823
4	Set screws (M5 x 6)	2826
5	Footplate screws (M5 x 8)	2827
6	Shoe mounting screws (M6 x 10)	2825

Accessories

Nr.	Item description	Art. nr.
7	Dorsal flexion component	2828

Basko Healthcare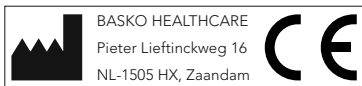

NEDERLAND

Tel.: +31 (0) 75 - 613 15 13
Fax: +31 (0) 75 - 612 63 73
E-mail: verkoop@basko.com

DEUTSCHLAND

Tel.: +49 (0) 40 - 85 41 87-0
Fax: +49 (0) 40 - 85 41 87-11
E-Mail: verkauf@basko.com

ÖSTERREICH

Tel.: +43 (0) 1 - 2 83 53 30
Fax: +43 (0) 1 - 2 83 62 62
E-Mail: verkauf@basko.com

basko.com