
Camp 2900

Hip Abduction Orthosis

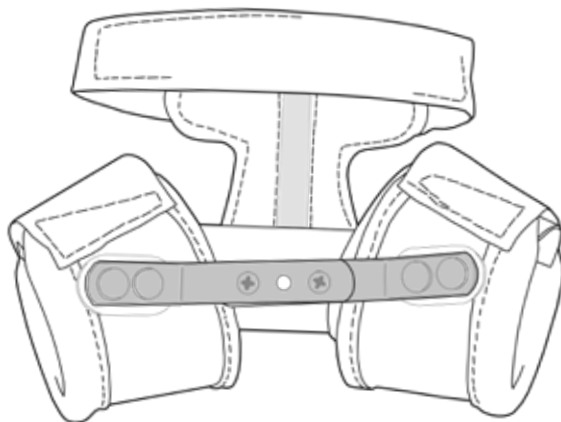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EN Instructions for use

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Basko
Healthcare

basko.com

NL	Gebruiksaanwijzing	4
DE	Gebrauchsanleitung	6
EN	Instructions for use	8
FR	Mode d'emploi	10

Camp 2900 Heupabductie orthese

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

Camp 2900 Heupabductie orthese

Fitting and adjusting the orthosis




Front side Camp 2900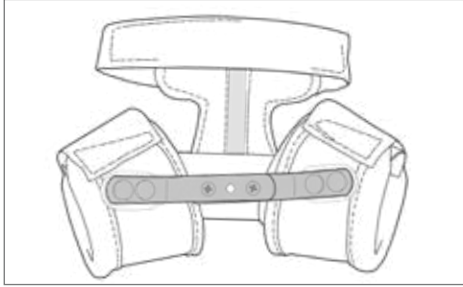



Back side Camp 2900



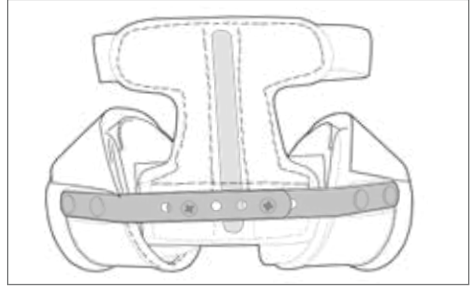
Step 1 | Lay the child on it's back and position the hips in 90° flexion (Fig. 1) and the legs each in 30° abduction (Fig. 2). Measure the distance between both knees (Fig. 3).



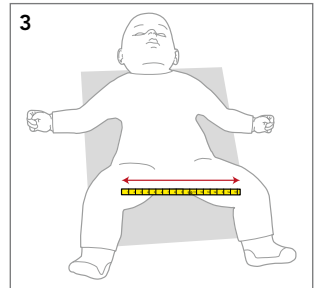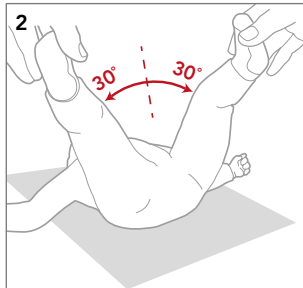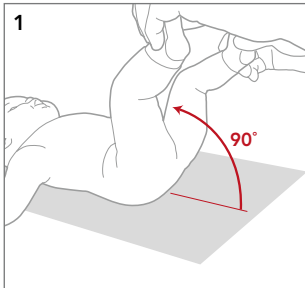
Step 2 | The width of the metal frame can be adjusted by unscrewing the Phillips screws (Fig. 4). Place the supplied Allen wrench on the flange nut on the inside of the frame for back pressure. Now adjust the frame to a width so the distance between the cuffs of the orthosis corresponds to the measured distance in step 1 (Fig. 5) where as a rule, there should be a maximum of one finger thickness of clearance between the outside of the cuff and the knee cavity. Replace the screws at the desired position and tighten firmly.

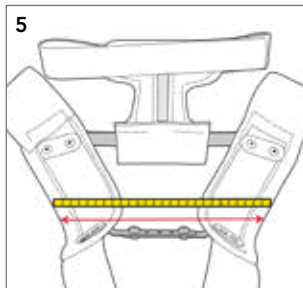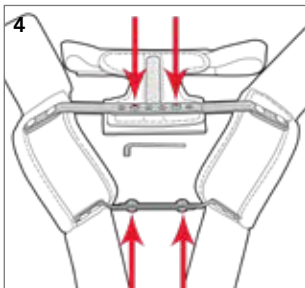Step 3 | Open the Velcro straps of the chest strap and both cuffs. Lay the child on its back and lift the lower back (Fig. 6) with a flat hand (not by the legs) and position the Orthosis so that the buttocks are flush with the bottom of the Orthosis (Fig. 7). First, place the cuff around the leg whose hip dysplasia is most severe (Fig. 8) and place the white artificial leather flap over the leg.

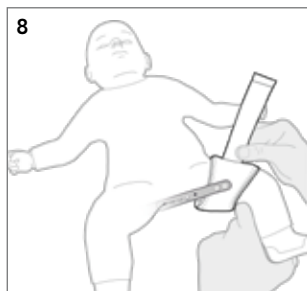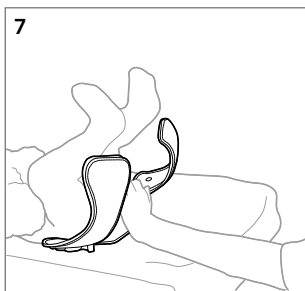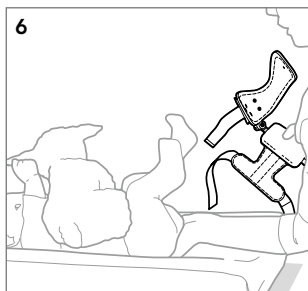Step 4 | Make sure the diaper is not tucked under the cuffs. Close the other cuff (Fig. 9) and then the strap around the tummy (Fig. 10). The band provides stability to the orthosis but should not be closed too tight. A finger should still fit between the abdomen and the strap without any difficulty (Fig. 11).

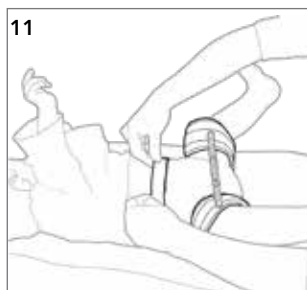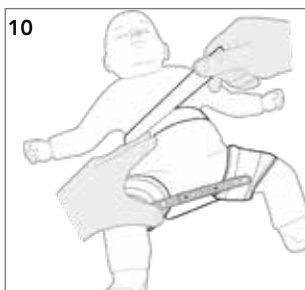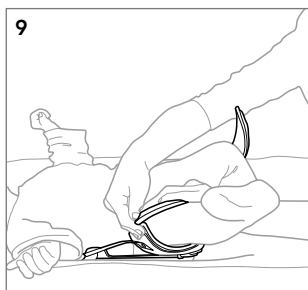Tips

- To transport your baby safely in the car, you can pad the car seat with towel rolls.
- When changing diapers, do not lift the baby by pulling its legs or feet.
- The thigh cuffs can fester, to prevent skin irritation you can put a "tube" of jersey fabric around them.
- When sleeping, the baby may find an easier position to lie in which may cause the orthosis to shift. If this happens, try moving the orthosis back so that the optimal spread position is achieved again. Especially during sleep, the hip recovers the most.

Washing Instructions

- Close all Velcro straps before washing to prevent damage to other garments and to maintain optimal adhesion of the Velcro straps. You can clean the orthosis with lukewarm water and mild soap..

CAUTION: Check the hook-and-loop fasteners regularly for contamination from dust and threads from other garments.

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