Basko
Healthcare

Clubycare by Basko Healthcare

NL	Gebruiksaanwijzing	2
DE	Gebrauchsanleitung.....	4
ENG	Manual	6
FR	Manuel	8
ES	Instrucciones para el usuario	10
IT	Istruzioni per l'uso	12
DK	Manual	14
SE	Manual	16
NO	Teknisk.....	18
FI	Käsikirja	20

Clubycare by Basko Healthcare

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Usage guidelines

Your doctor prescribed the orthotic and determined how long it should be used. It is important to comply strictly with the daily usage prescribed by the doctor. Medicinal ointments, etc., should only be used under the supervision of a doctor.

The technical staff who fitted the shoes are responsible for ensuring that the shoes fit perfectly and function properly. In the event of skin irritation, unusual pain or other complications that you feel are caused by the product, contact your doctor and/or the technical staff.

Intended use of the product/Indications

Clubycare shoes are used for treating club feet, according to, among others, the

„Ponseti“ method. The Clubycare shoe is an orthopaedic component. The complete orthotic is comprised of a splint (P.E.A.care) onto the ends of which the Clubycare shoes are mounted. The orthotic provides the necessary external rotation of the feet and positioning according to the width of the shoulders. This maintains the corrected foot (/feet) position as determined in initial treatment and prevents a relapse.

The shoe is used for the following indication:

- Pes equinovarus congenitus (ICD10: Q66.0)

Extra product information/field of use

The shoe is not suitable for use in sea (salt) water.

Inherent risks of use and contraindications

The shoes may not be worn on pressure points or on irritated skin. There are no other known risks of use or contraindications.

Care advice

Check the shoes regularly for damage on the inside, the fastenings and the screw receptacles (underside of shoes).

Material specifications

Shoe: leather; sole: synthetic

Cleaning instructions/disinfecting instructions

Clean the shoe with a cloth and water no warmer than 40°C. Do not bleach. Do not dry-clean. Not suitable for tumble-drying.

Instructions

Check the packaging immediately:

A) 1 pc Clubycare shoe

B) Instructions for use

Putting on the shoe

Step 1: Measure the length and the breadth of the foot and refer to the size chart in order to select the right size (see size chart).

Step 2: Open the shoe's fastening, band and laces/velcro (*Fig. 1*).

Step 3: Position the foot in the shoe. Make sure the heel is positioned properly towards the back of the shoe. Check for correct positioning via the heel cut-out (*Fig. 2*).

Step 4: Place the strap over the instep, pull it to the back and fasten it. Check again that the foot is properly positioned towards the back (*Fig. 3*).

Step 5: Pull the top and bottom of the sock or stocking tight so there are no folds inside the shoe. (*Fig. 4*).

Step 6: Fasten the lace or velcro and repeat steps 1 to 4 with the other foot and shoe (*Fig. 5*).

Step 7:

(only in combination with the P.E.A.care splint)

To mount the shoes on the splint, slide the shoe with the foot plate sideways onto the splint. Press and click the foot plate into position on the adapter. To separate them, press the red release clamps together and slide the foot plate off the adapter (*Fig. 6*).

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